

A superficial vein, most commonly used for venipuncture, it lies over the cubital fossa and serves as an anastomosis between the cephalic and basilic veins.

## 2. Cephalic vein

Shown in both forearm and arm, it can be followed proximally where it empties into the axillary vein.

### 3. Basilic vein

Shown in the forearm and arm, it divides to join the brachial vein.

#### **BD Diagnostics**

Preanalytical Systems

BD Global Technical Services 1.800.631.0174 BD Customer Service 1.888.237.2762 www.bd.com/yacutainer

# **Inappropriate Sites** for Venipuncture

- Arm on side of mastectomy
- Edematous areas
- Hematomas
- Arm in which blood is being transfused
- Scarred areas
- Arms with fistulas or vascular grafts
- Sites above an IV cannula

Follow your facility's phlebotomy procedures.

# **BD Diagnostics**Preanalytical Systems

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2004 BD. 6/04 VS5730-2 www.bd.com/vacutainer